

Trilogy200




Quick start guide

- 1 Overview
- 2 Circuit set-up
- 3 Set-up screen
- 4 Prescription set-up

1

Overview

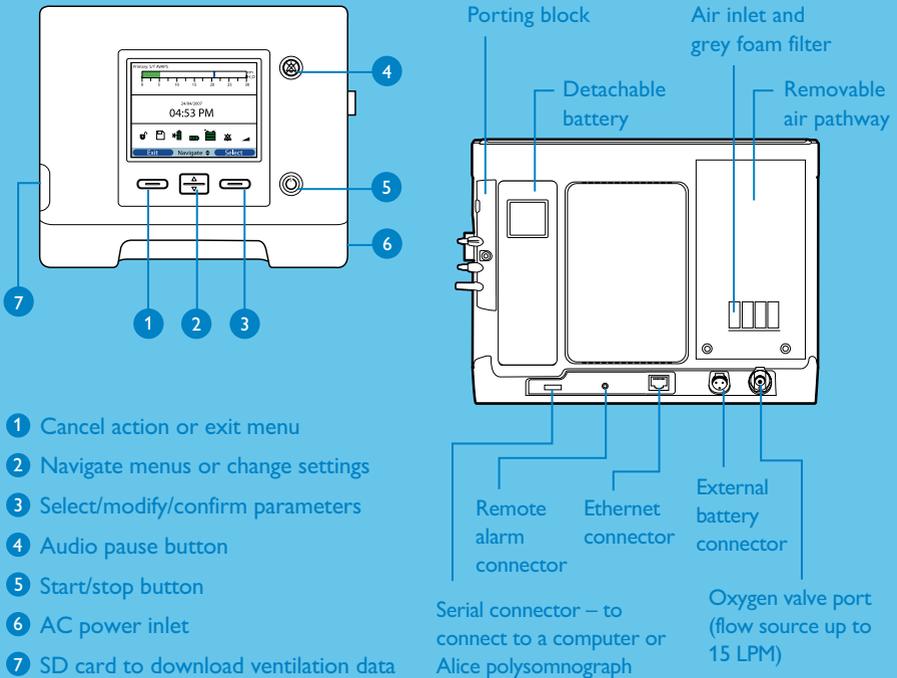

Symbols and icons



Prescriber environment, full access mode



SD card inserted in the ventilator



Audio pause active



Ramp active



Status of detachable battery



Status of internal battery



Status of external battery




Battery is charging

2

Circuit set-up


Before operating Trility200, you need to configure the circuit type.

Passive exhalation port circuit

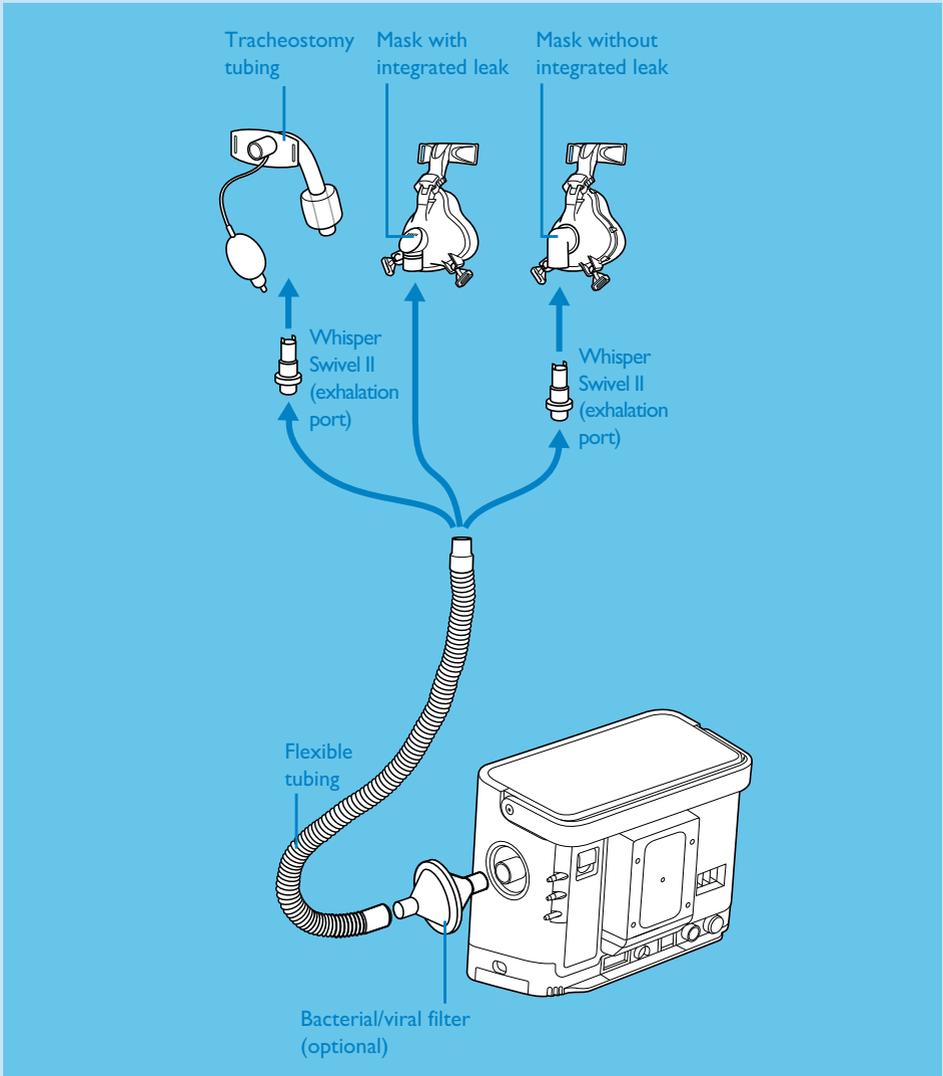


In the Set-up menu, select "Passive" for circuit type.

Active exhalation valve with proximal airway pressure circuit

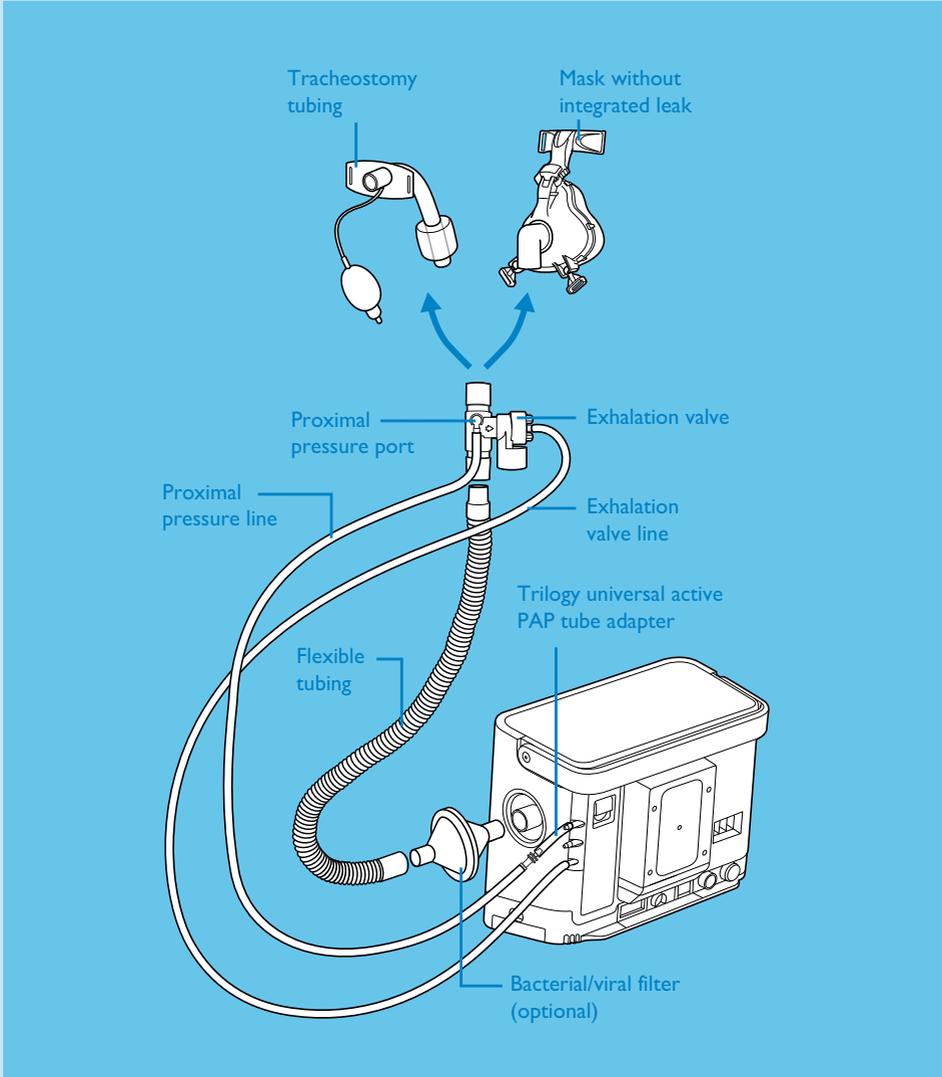

In the Set-up menu, select "Active PAP" for circuit type.

Active exhalation valve flow sensor circuit

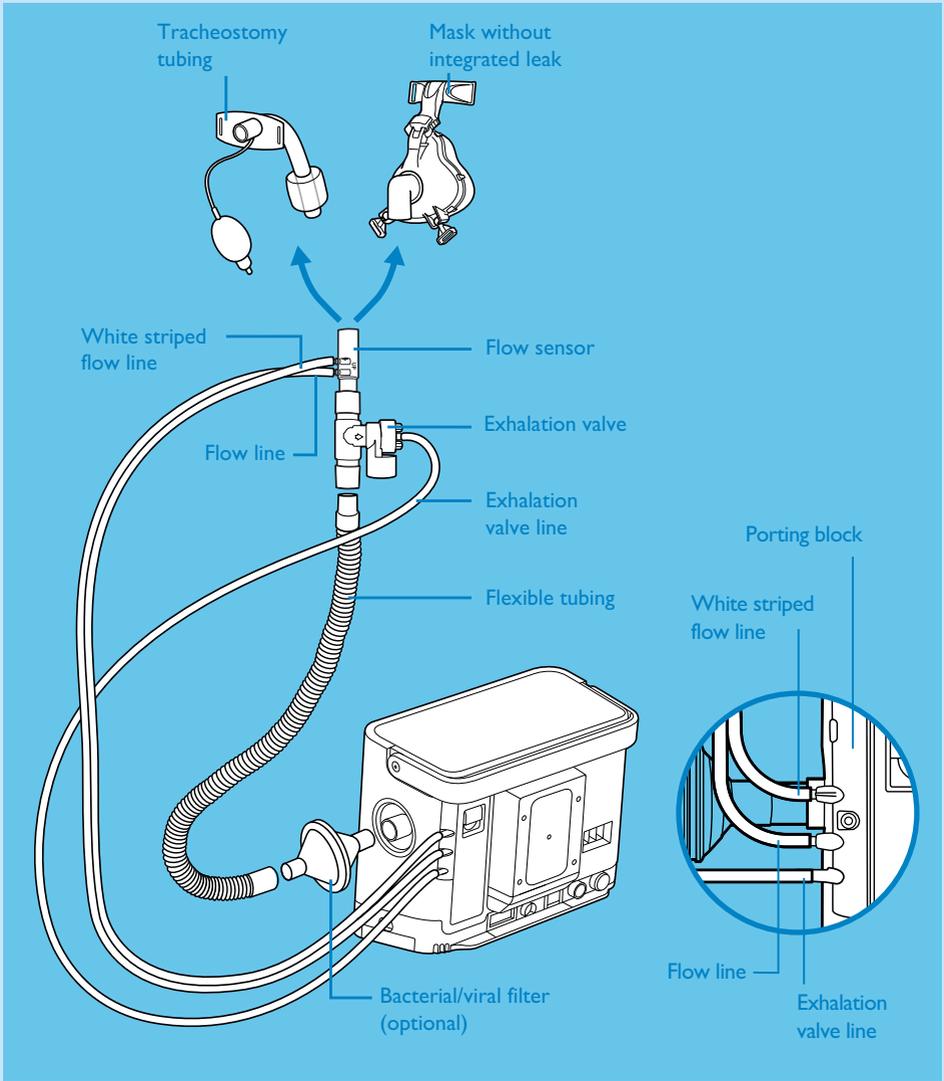


In the Set-up menu, select "Active flow" for circuit type.

3


Set-up screen

Setting the circuit type


Make sure Trilogy200 is turned off before taking the following steps

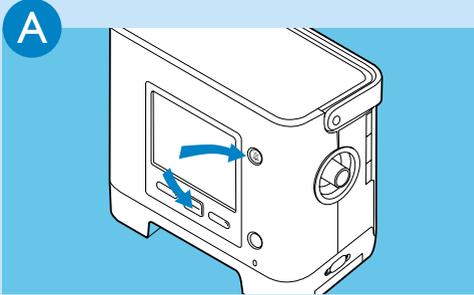


Press and hold  and  for a few seconds to access **Set-up** page

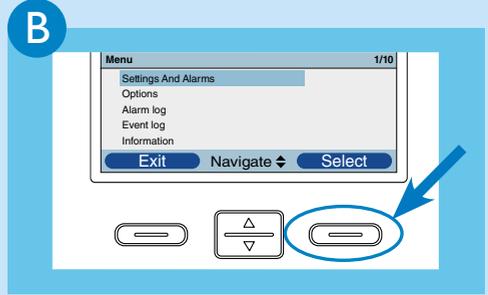

Select **Settings And Alarms**

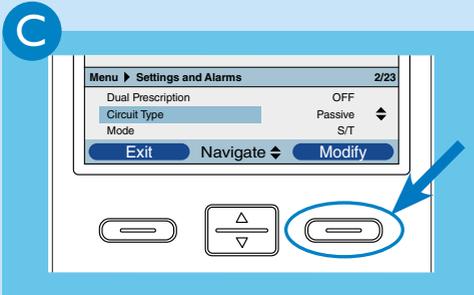


Navigate to **Circuit Type** and press 

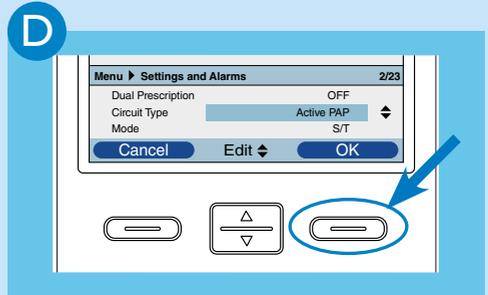


Use  to choose **Active PAP**, **Active flow**, or **Passive** and then press  twice to confirm

Setting menu access


To enable full access to the settings

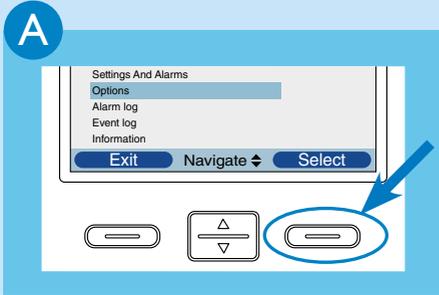


Navigate to **Options** and press **Select**

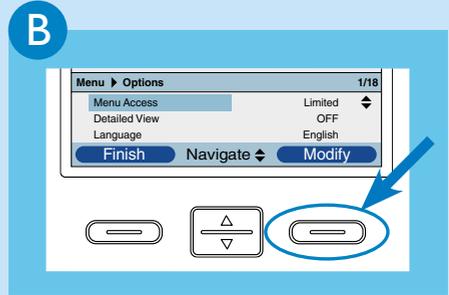


Press **Modify** to enter **Menu Access**

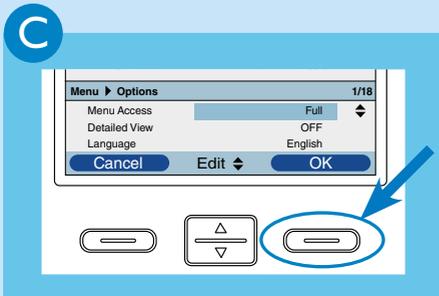

Use  to choose **Full** or **Limited Access** and press **OK** to confirm

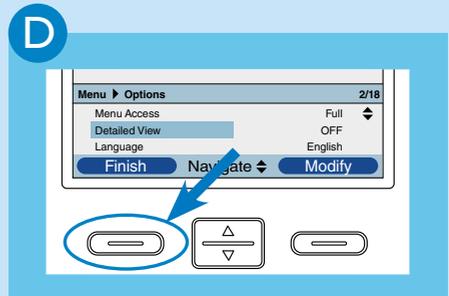


Press **Finish** and **Exit** to return to **Initial Screen**

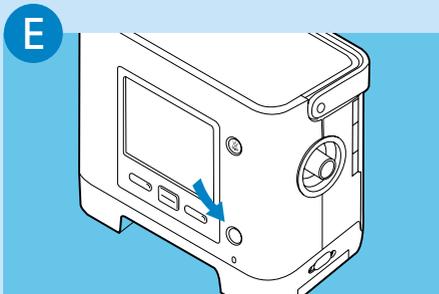


Press  to begin therapy **or** continue navigating through the menu to set-up the ventilation parameters

4


Prescription set-up

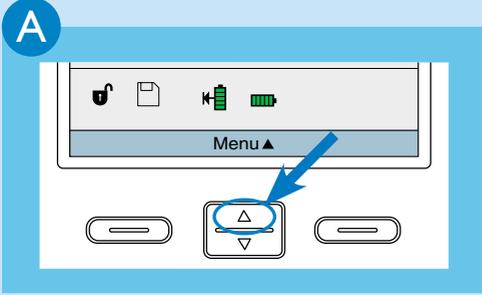


From main screen, press **Menu**

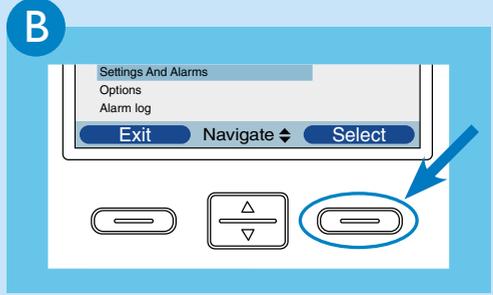

Select **Settings And Alarms**

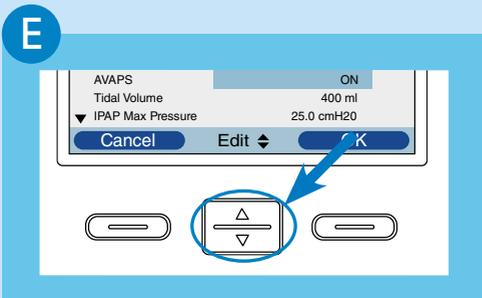


Use  to change the value or status of setting

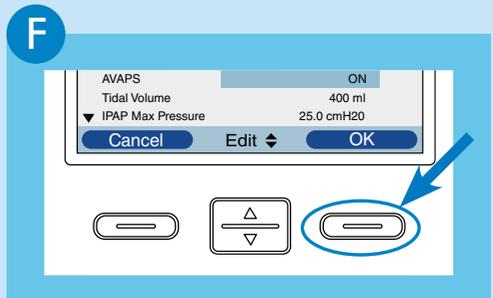


Press **OK** to validate new setting


Dual Prescription

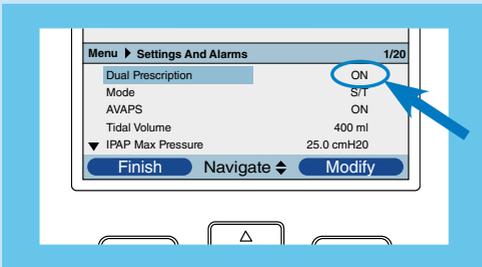


If **Dual Prescription** is enabled, both prescriptions need to be adjusted as previously explained

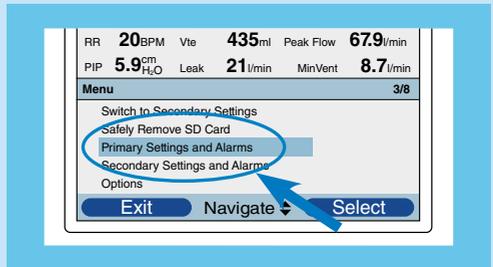


Two prescriptions to set

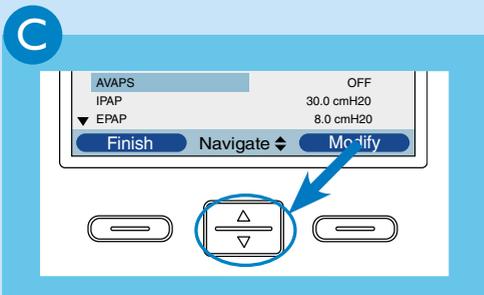

Scroll up and down using  to navigate through the settings and alarms

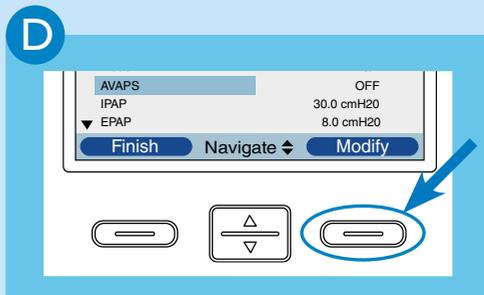

Press **Modify** to change the parameter highlighted in blue

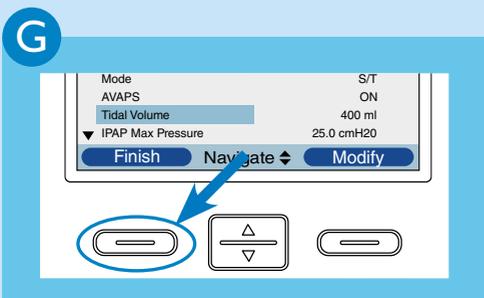

Press **Finish** and **Exit** to return to the main screen

**Philips Healthcare is part of
Royal Philips Electronics**

How to reach us

www.philips.com/healthcare
healthcare@philips.com
fax: +31 40 27 64 887

Asia

+852 2821 5888

Europe, Middle East, Africa

+49 7031 463 2254

Latin America

+55 11 2125 0744

North America

+1 425 487 7000

800 285 5585

(toll free, US only)

Philips Respironics
1010 Murry Ridge Lane
Murrysville, PA 15668

Customer Service
+1 724 387 4000
800 345 6443
(toll free, US only)

Respironics Asia Pacific
+65 6298 1088

Respironics Australia
+61 (2) 9666 4444

Respironics Deutschland
+49 8152 93 06 0

Respironics Europe,
Middle East, Africa
+33 1 47 52 30 00

Respironics France
+33 2 51 89 36 00

Respironics Italy
+39 03 62 63 43 1

Respironics Sweden
+46 8 120 45 900

Respironics Switzerland
+41 6 27 45 17 50

Respironics United Kingdom
+44 800 1300 845
www.philips.com/respironics

Please visit www.philips.com

Respironics, Trilogy, Whisper Swivel, and Alice are trademarks of Respironics, Inc. and its affiliates. All rights reserved.



© 2009 Koninklijke Philips Electronics N.V. All rights are reserved.

Philips Healthcare reserves the right to make changes in specifications and/or to discontinue any product at any time without notice or obligation and will not be liable for any consequences resulting from the use of this publication.

Geyer WMB 11/09/09 MCI 4102961 PN 1068897